

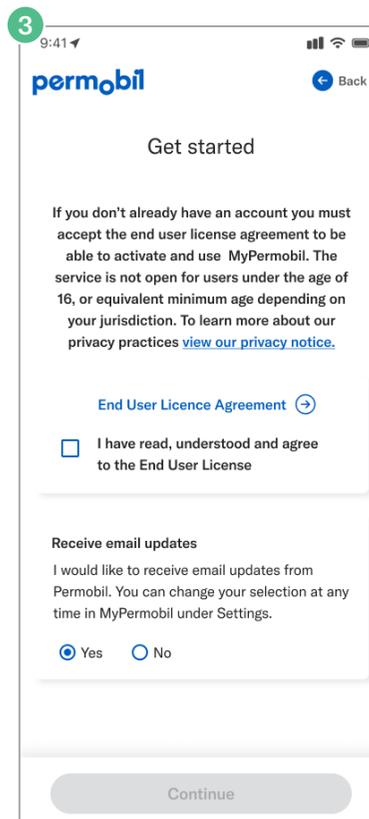


MyPermobil—Activation guide

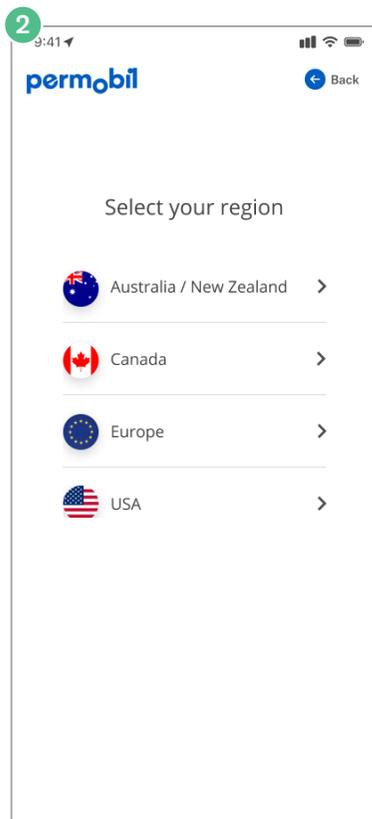

Note: Wi-Fi is required for set-up

Getting started

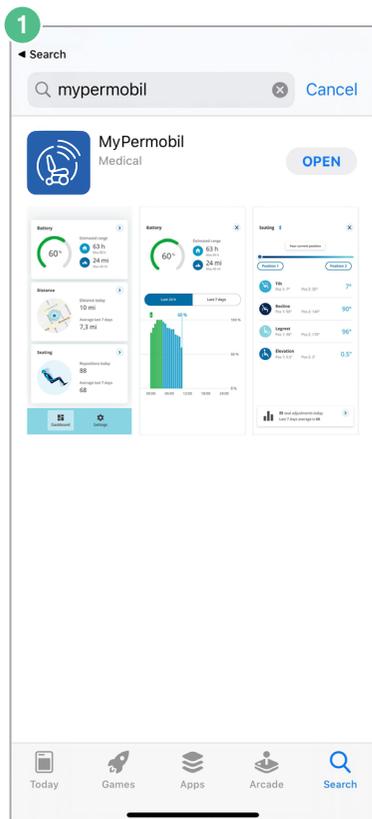

Confirm with provider that wheelchair was ordered with Permobil Connect.

Choose your region.

Read/accept MyPermobil policy.

Download MyPermobil on your Android or Apple phone.


MyPermobil

MyPermobil connectivity is an available option on all new M and F-Series Corpus & Corpus VS models. Be sure to ask for Permobil Connect at time of order!


App Store is a trademark of Apple Inc. Google Play is a trademark of Google LLC.

Note: Wi-Fi is required for set-up


Create account

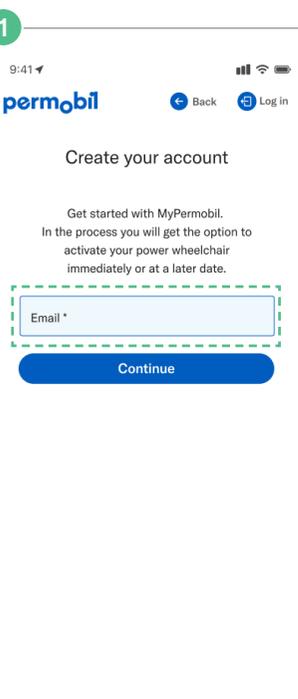

Enter e-mail address.

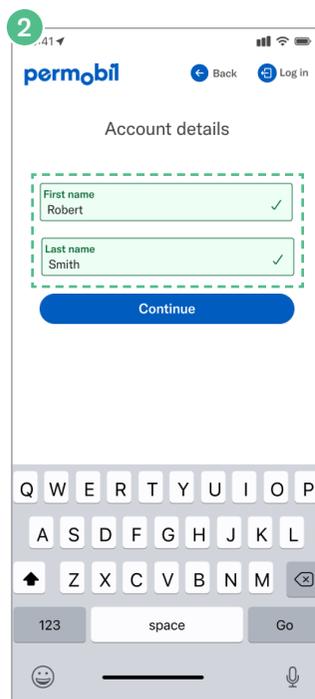

Enter your name.

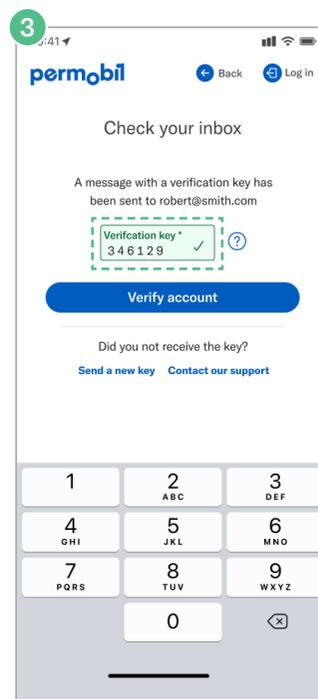


Check email, click “Verify” in email (if opened on same device as MyPermobil app) and follow prompts


- OR -

Enter verification code found in email, tap “Verify account”.


Activate

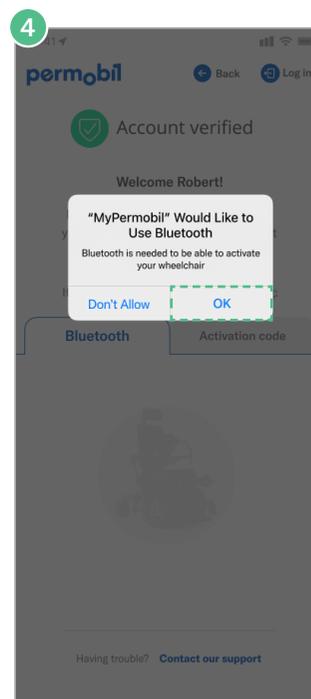

Accept to allow Bluetooth™, and turn on your wheelchair.

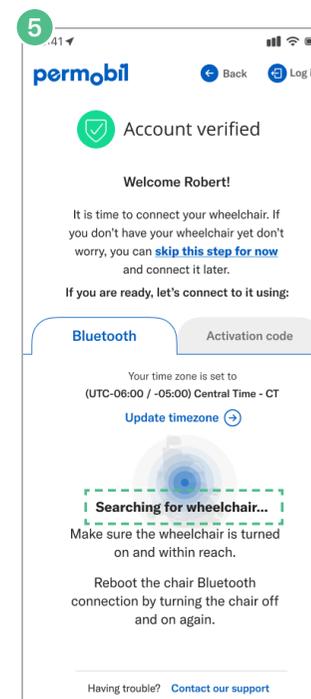

You may need to turn your wheelchair off and on again in order to reboot your chair’s Bluetooth connection.

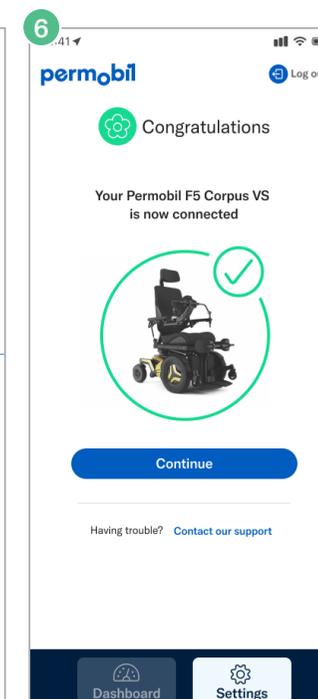

You are connected!