



Cleaning a ROHO® DRY FLOATATION® cushion and cover

Clean your ROHO cushion regularly to maintain hygiene and ensure your needs are met throughout the lifetime of the cushion. Be sure to follow manufacturer safety guidelines for all chemicals. **Important!** Cleaning and disinfecting are separate processes. Be sure to *clean first, then disinfect*.

1 Cleaning the ROHO Cushion Cover

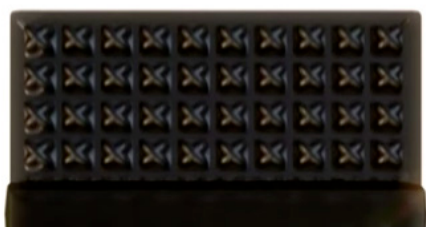

Remove the cushion from the cover. If you have the optional hook and loop fasteners attached, remove them from the cover.

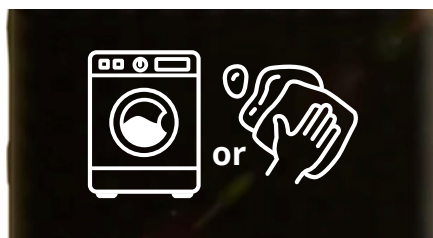


Machine wash on the gentle cycle in cold water with mild detergent, or wash by hand with neutral detergent and warm water.




Tumble dry on low or no heat, or hang to air dry.


2 Disinfecting the ROHO Cushion Cover

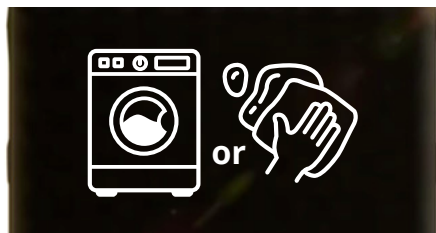

Machine wash on the gentle cycle in hot water with mild detergent, or hand wash in warm water with a mixture of 9 parts water: 1 part bleach.

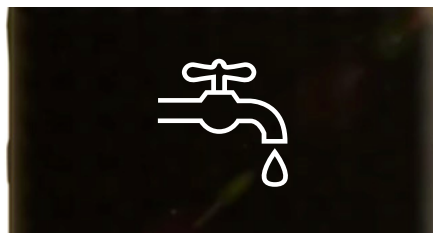

Rinse with fresh water.

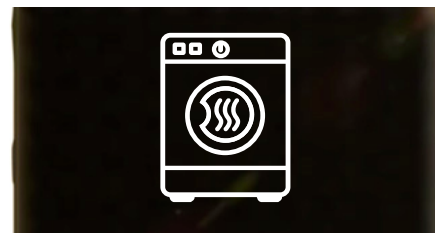

Tumble dry on low or no heat, or hang to air dry.


3 Cleaning the ROHO Cushion



Close the valve(s).
For Sensor Ready Cushions, if any devices such as Smart Check are attached, make sure it's removed before starting cushion cleaning, as water in cushion can result in damage and false readings with these devices.



Begin by rinsing and removing anything between the cushion cells.
Deflating air cells may make it easier to thoroughly clean the cushion.

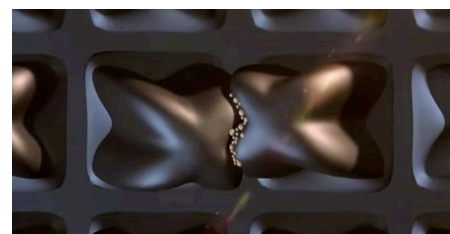


Take care while cleaning. If a substance has dried on the cushion and sticks cells together, do not attempt to separate the cells when they are dry. This may damage the cushion.




Check the valve(s) to make sure it is still closed. Place in a large sink. A clean shower or a tub is also acceptable. Do not allow water or cleaning solution to enter the product.



Hand wash the cushion using a mild dishwashing liquid or laundry detergent.

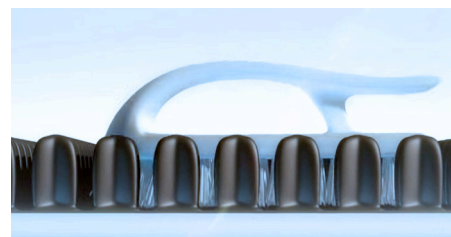

Use a soft plastic bristle brush, sponge or washcloth to gently scrub all cushion surfaces.

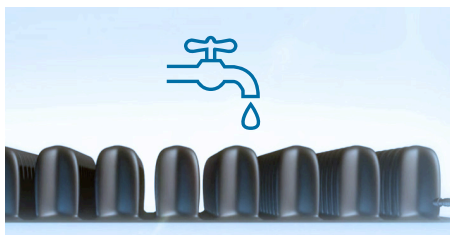

Rinse with fresh water.



Hang the cushion to air dry completely.

4 Disinfecting the ROHO Cushion




Repeat previous cleaning instructions using a solution of 9 parts warm water: 1 part household bleach. Most germicidal disinfectants are safe if the manufacturer's dilution directions are being followed.




Keep the clean product wet with bleach solution for 10 minutes, then rinse thoroughly with fresh water.



Hang the cushion to air dry completely.

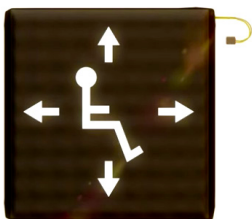

Once the cushion and cover are both dry, replace the cushion inside the cover. Use the directional labels on the side of the cushion cover to be sure the cushion is inserted correctly.

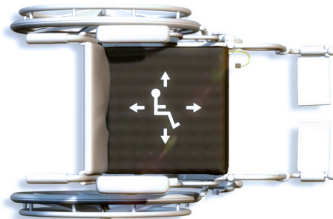

Zip the cover and place the cushion on the wheelchair. Again, refer to the directional labels on the side of the cushion cover to be sure it is placed properly on the wheelchair.

Note: Follow cushion setup instructions in the product operation manual provided with the cushion.

For more detailed information on cleaning and disinfecting a ROHO cushion, additional instructions, cautions and information regarding sterilisation refer to the product operation manual provided with your cushion.

The following are trademarks and registered trademarks of Permobil ROHO Seating and Positioning: DRY FLOATATION® and ROHO®.

RH-DF-CG-01.0