


MyPermobil—Activation guide


Note: Wi-Fi is required for set-up

Getting started

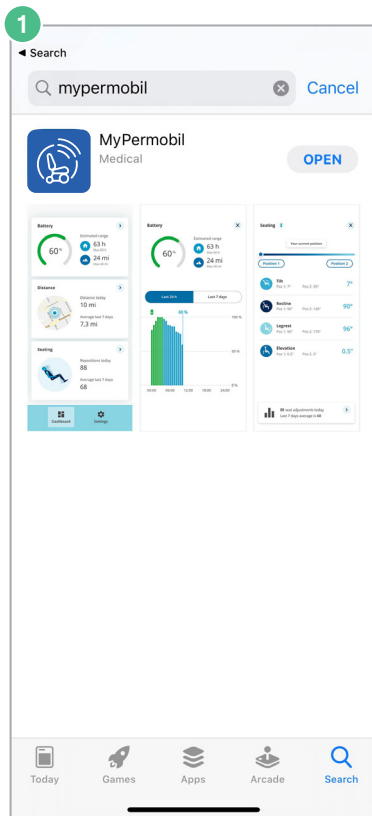


Confirm with provider that wheelchair was ordered with Permobil Connect


Download MyPermobil on your Android or Apple phone

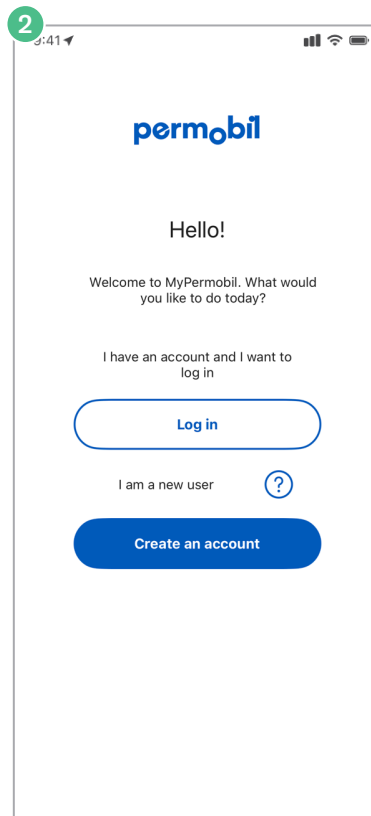

Click on activate a new wheelchair

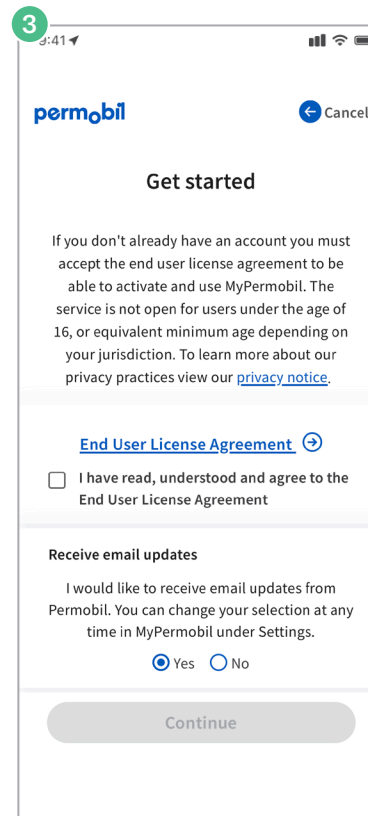

Read/accept MyPermobil policy

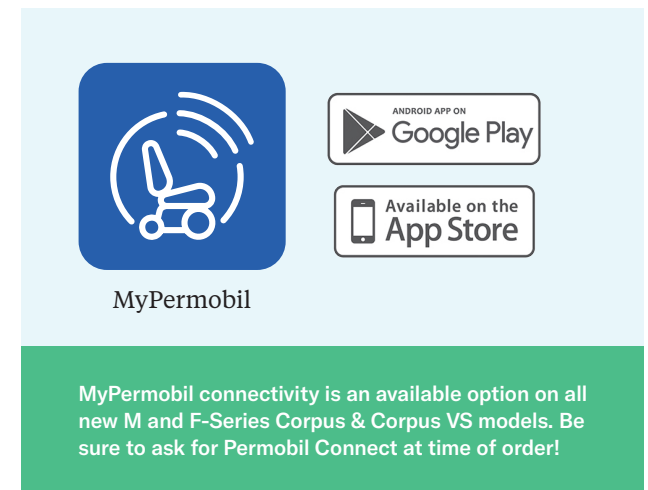

MyPermobil—Activation guide

Note: Wi-Fi is required for set-up

Create account

1

9:41

permobil

Back Log in

Create your account

Get started with MyPermobil. In the process you will get the option to activate your power wheelchair immediately or at a later date.

Email *

Continue

Enter e-mail address

2

9:41

permobil

Back Log in

Account details

First name Robert

Last name Smith

Continue

Enter your name

3

9:41

permobil

Back Log in

Check your inbox

A message with a verification key has been sent to robert@smith.com

Verification key * 346129

Verify account

Check email, click “Verify” in email (if opened on same device as MyPermobil app) and follow prompts

- OR -

Enter verification code found in email, tap “Verify account”

Activate

4

9:41

permobil

Back Log in

Account verified

Welcome Robert!

“MyPermobil” Would Like to Use Bluetooth

Bluetooth is needed to be able to activate your wheelchair

Don't Allow OK

Accept to allow Bluetooth™, and turn on your wheelchair

5

9:41

permobil

Cancel Log in

Activate your wheelchair

Bluetooth Activation code

Your time zone is set to (UTC -06:00 / -05:00) Central Time - CT

Select time zone

Searching for wheelchair...

Make sure the wheelchair is turned on and within reach. Reboot the Bluetooth connection by turning the wheelchair off and on 3 times.

Having trouble? Contact Permobil support

You may need to turn your wheelchair off and on 3 times in order to reboot your chair's Bluetooth connection

6

9:41

permobil

Log out

Congratulations

Your Permobil F5 Corpus VS is now connected

Continue

Having trouble? Contact our support

Dashboard Settings

You are connected!