

Charging guide

Your wheelchair is equipped with maintenance-free AGM batteries from Permobil. This means that you need to charge the batteries regularly; no other maintenance is required. It is recommended to charge your wheelchair every evening/night when you have finished driving for the day and it is parked.

To charge the wheelchair battery, connect the charger plug to the charging socket located on the joystick, control panel, or external charging port. Then, plug the charger into a wall outlet. The charging process will begin automatically.

For optimal performance, charge and store the wheelchair in a normal indoor temperature of approximately 20°C. In colder conditions, charging may take longer, and battery capacity can be reduced, potentially decreasing the travel distance per charge.



The charger operates automatically and does not need to be monitored. It should not be connected to a socket with a timer. When charging is complete, all four lights on the charger will illuminate. You can leave the charger connected until you next use your wheelchair.

To finish charging, first disconnect the plug from the wall socket and then the charging plug from your wheelchair.



NOTE! Charging should normally last at least 8 hours. Fully discharged batteries require approximately 12 hours to reach a full charge. Do not interrupt the charging process until it is complete and all four lights on the charger are illuminated.

If the wheelchair is rarely used and the battery level rarely drops below 50%, you should once a week use the chair more intensively so that the battery level reaches the red zone before charging.

