

Permobil

Colored lights package

The colored light package is now available on all power platform powered wheelchairs. If you need help updating your wheelchair for the colored light package, please contact your local Permobil sales representative or our Customer Experience team at 1-800-736-0925.

Requires at least firmware 1.57 and a configuration package for the colored lights.

Once the packages are sent to your chair, you may see notifications on the display indicating steps of the installation process.


MyPermobil app:

1. Install or Update MyPermobil app to the latest version (must be at least 1.75)
2. Complete steps to create an account and activate your chair (if not previously done) or if not Factory-Connected as of 4/28/26 for chairs delivered in the US. See video [here](#).
3. Ensure chair is in BT pairing mode (see video [here](#))
4. Once chair is activated and connected to the MyPermobil app, go to the *Settings* tab.
5. Select *My Wheelchair* from the menu.
6. Select *Wheelchair Settings*
7. Select *Colored Lights*
8. Toggle colored lights ON (*This button will turn colored lights off and back on without requiring the joystick or an external switch*).
9. Use sliders to select Color, Color intensity, & Brightness.
10. Choose *Set Color* and the lights will change to the color selected within


Note: The switch for the lights will function as follows when colored lights are programmed.

- Press #1 - Normal lights
- Press #2 - Colored lights
- Press #3 - Lights off



Step 4

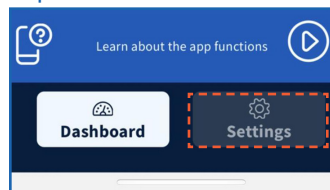


Step 6

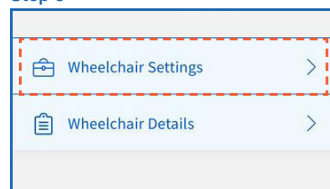


Step 8

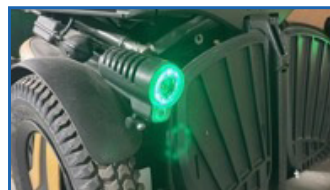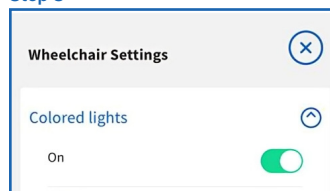

Step 5

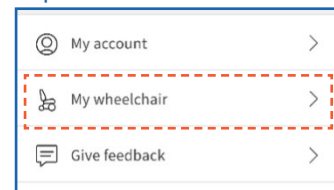

Step 7

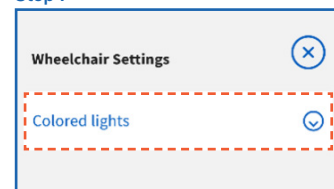

Step 9 & 10

