


# Factory Reset Instructions for PushTracker E2/E3

**NOTE:** Wi-Fi is required for Wear OS by Google set-up. If doing a Wear OS by Google set-up on a PushTracker E2/E3 that was preconfigured by Permobil, you must do a factory reset first.


## 1 Factory Reset PushTracker E2/E3



1. Wake up the screen.
2. Tap side button (top one for E3) to go into the App Menu.
3. Tap the **Settings** icon.

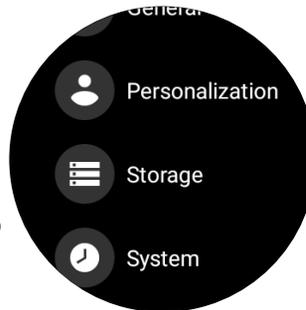

4. Select **System**

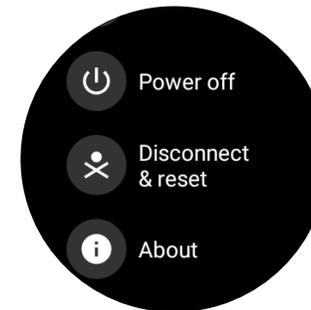

5. Tap **Disconnect & Reset.**
6. Scroll down and tap the blue check mark to start the process.
7. Wait until the process is complete.

**NOTE:** Note: Factory resetting your PushTracker E2 or E3 will delete all data, including the SmartDrive MX2+ app, and you will not be able to use your PushTracker with a SmartDrive until all of the additional steps in this document are completed.

For more instructions check out the PushTracker E2 how-to videos at [permobilus.com/howtosmartdrive](https://permobilus.com/howtosmartdrive).

# PushTracker E2/E3 — Wear OS by Google set-up

**NOTE:** Wi-Fi is required for Wear OS by Google set-up. If doing a Wear OS by Google set-up on a PushTracker E2/E3 that was preconfigured by Permobil, you must do a factory reset first.

## 2 PushTracker E2/E3 & Wear OS by Google

1. Download Wear OS by Google on your Android or Apple phone



2. Read/accept Google terms of service & enable Bluetooth® sharing

3. Turn on watch, select language

4. Read/accept Mobvoi policy

5. Pair/connect PushTracker E2/E3 to phone






6. Connect your watch to Wi-Fi

7. Choose/sign in/create Google account

8. Initialization

9. Wear OS tutorial

- Smartphone
- PushTracker E2/E3

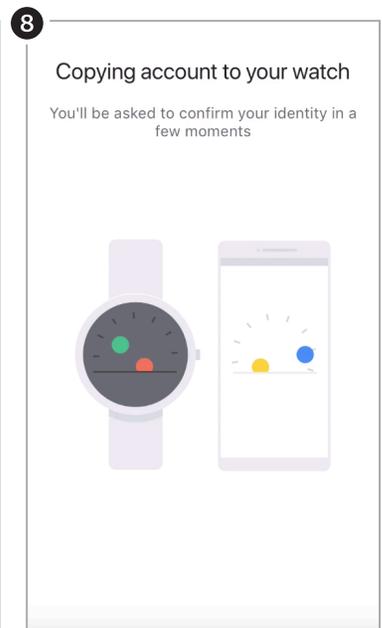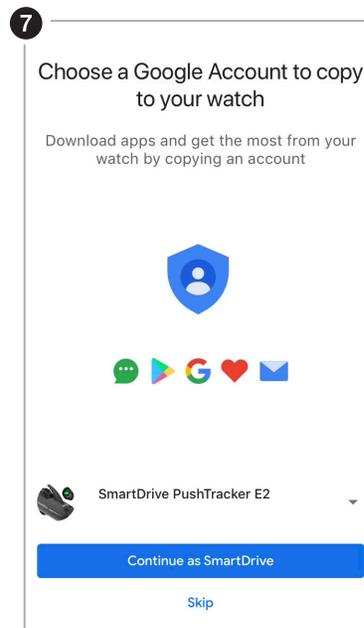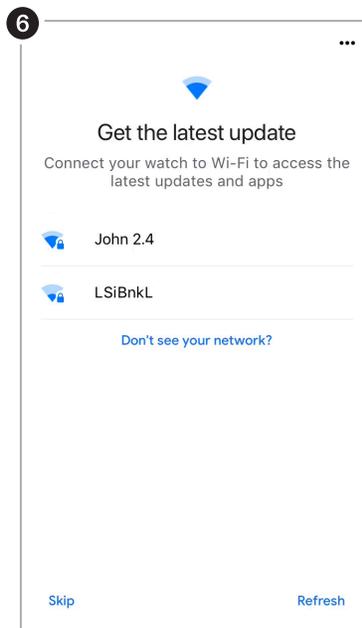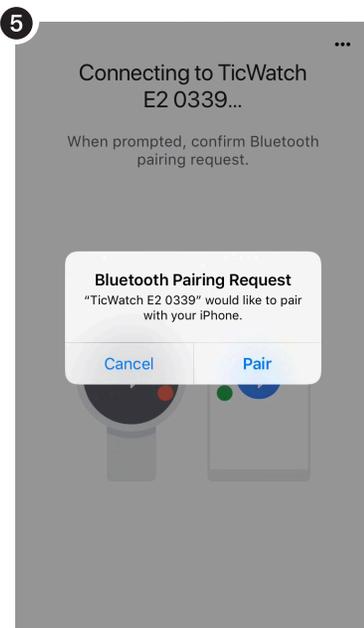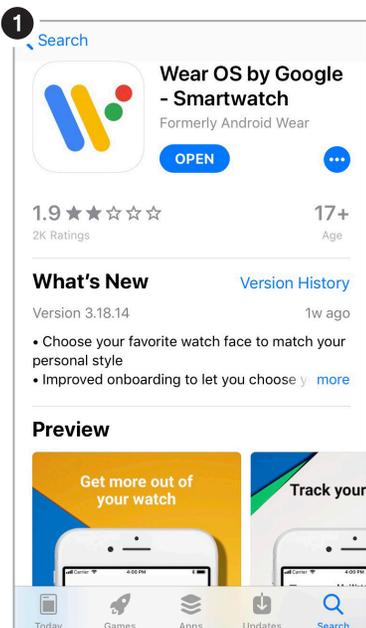

**NOTE:** Wi-Fi is required for Wear OS by Google set-up. If doing a Wear OS by Google set-up on a PushTracker E2/E3 that was preconfigured by Permobil, you must do a factory reset first.

## 3 Download SmartDrive MX2+ app



1. Go to Google Play Store on PushTracker E2/E3



2. Search for "SmartDrive MX2+"




3. Install & launch SmartDrive MX2+ app


## 4 Pair & go



1. With your SmartDrive MX2+ unit turned on, launch the SmartDrive MX2+ app and touch "TURN POWER ASSIST ON"

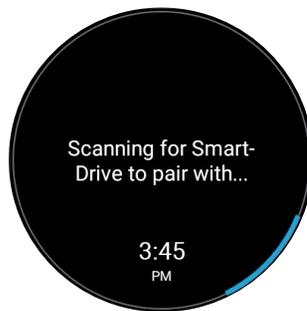

2. Pair with SmartDrive MX2+ by selecting the number listed on the screen. You can only pair when one SmartDrive MX2+ is present

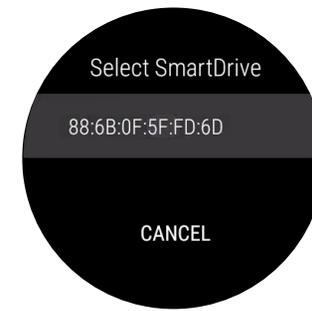

3. You are now paired & connected!

**NOTE:** You can also pair to your SmartDrive MX2+ through the settings menu within the app.

For more instructions check out the PushTracker how-to videos at [permobilus.com/howtosmartdrive](https://permobilus.com/howtosmartdrive).



\*Wear OS by Google, iPhone, Android, Bluetooth and Google Play are trademarks or registered trademarks of their respective owners.